



# GO<sub>2</sub>VENT<sup>®</sup> Surgical Center Kit

A Must Have for All Ambulatory Surgery Centers!

- Meets CMS "Emergency Equipment" 416.44(d) mandate for mechanical ventilatory assistance equipment
- Meets FDA, ASTM, and ISO guidelines for gas-powered, automatic resuscitators
- Disposable ventilator for short-term emergency backup ventilation
- APM-Plus airway pressure monitor for real-time respiratory monitoring and backup alarms
- Easy setup with the training manuals
- Kit includes (Order No. 5050):
  - 3 ea - GO<sub>2</sub>VENT<sup>®</sup> (Model GCM-6123)
  - 1 ea - Oxygen Regulator (RG-2162)
  - 1 ea - APM-Plus (Model 3960)
  - 1 ea - Test Lung

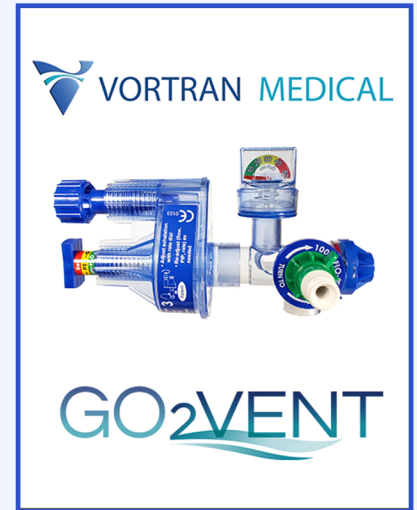

## QUICK GUIDE

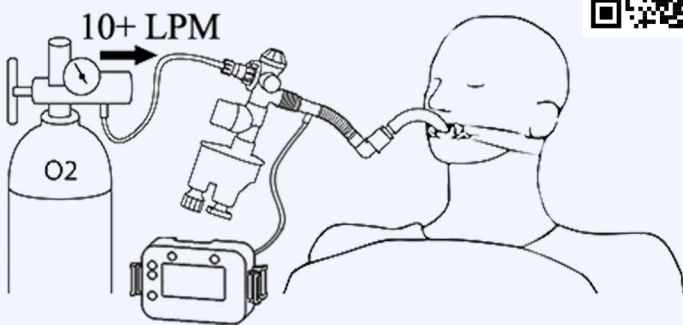

1. Connect GO<sub>2</sub>VENT and APM-Plus
2. Supply O<sub>2</sub> ≥10 LPM
3. Adjust FiO<sub>2</sub>, PIP / PEEP, Respiratory Rate (RR)



### ALARMS:

- High PIP
- Low PEEP
- High RR
- Non-Cycling



### MONITOR:


- PIP • I-Time
- PEEP • E-Time
- RR • I:E



2 X AA



On/Off

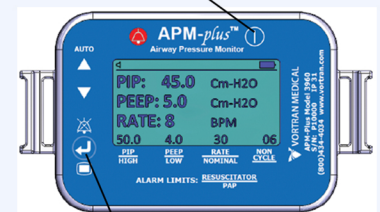

≡ Hold for Alarm Limits Menu